# OPERATION MANUAL

Please read this OPERATION MANUAL carefully before use, and file for future reference.

# NSK Sterile Cutting Accessories

Rx Only

OM-Z0147E 004

Thank you for purchasing NSK Sterile Burs/Drills/Discs/Blades/Rasps. Read this manual carefully, and keep this Operation Manual in a place where a user can refer to it at any given time. The products should be used with an appropriate Attachment/Handpiece as specified in the Operation Manual for use for each product.

In this Operation Manual, Bur, Drill, Disc, Blade and Rasp are collectively called Cutting Accessory(ies).

## 1 User and Indications for Use

User: Qualified Professionals (Surgeon with knowledge and skills required for operating this product).

Indications for Use: NSK Sterile Cutting Accessory(ies) is for single use only. NSK Sterile Cutting Accessory(ies) is intended for use in bone surgery or cutting implant (The applicable Cutting Accessory(ies) for cutting implants is "Primado Metal Cutting Carbide Bur" and "Primado Metal Cutting Diamond Disc". Do not use other Cutting Accessories to cut implants).

## 2 Precautions for handling and operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 <b>WARNING</b>	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.
 <b>CAUTION</b>	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.

## WARNING

- Cutting Accessory is for single use only. Do not reuse or resterilize. If you reuse or resterilize, the following risks may occur.
  1. Infection.
  2. Personal injury due to damaged Cutting Accessory.
  3. System defect due to damaged/deformed Cutting Accessory.
- When mounting or removing a Cutting Accessory, be sure to turn off the Safety Lock on the Hand Switch of the Motor, or completely remove your foot from the Foot Control Pedal to stop the Attachment completely. Otherwise, a patient or surgical staff may be injured by unintentional movement of the Motor.
- Check there are no abnormalities of appearance on the package. DO NOT use if the pouch is damaged.
- In order to prevent necrosis of bones and tissues, or thermal injury, use appropriate irrigation while cutting.

**WARNING**

- DO NOT use Cutting Accessories with problems listed below. Otherwise, it may prevent operation and may cause damage to bones and tissues or a thermal injury due to heat generation.
  1. Bended, deformed, rusted, broken, cracked, worn, deficient, scratched Cutting Accessories.
  2. Modified Cutting Accessories.
- DO NOT place the Cutting Accessory/Attachment/Handpiece/Motor on the patient or the drape covering the patient. Failure to do so may cause an accident, injury or thermal injury to the patient and/or surgical staff.
- DO NOT put excessive pressure on the "Cutting Accessory", which may bend or damage the "Cutting Accessory". Excessive pressure may cause heat generation on the "Cutting Accessory", Attachment/Handpiece and Motor which could lead to thermal injury. Use "Cutting Accessory", Attachments/Handpieces and Motors intermittently to prevent heat generation. In case of heat generation, immediately stop using until it cools down. DO NOT touch the heated area to prevent thermal injury.

**CAUTION**

- Be sure to use the "Cutting Accessories" to NSK specified Attachment/Handpiece.
- These are medical system/products; do not allow anybody to use it other than qualified professionals who are familiar with the procedures. Use these products in accordance with these indications for use and proper method of use.
- In order to avoid system damages due to the incorrect assembly, operation and/or improper handling, use Cutting Accessories only in the ways specified in this manual and each Handpiece/Attachment Operation Manual.
- DO NOT modify, reshape or repair Cutting Accessories under any circumstances.

## CAUTION

- Prior to Cutting Accessory usage, check and ensure correct operational condition and physical appearance. DO NOT use if damaged in any way.
- If the Cutting Accessory is broken during an operation, be sure to remove all of the broken pieces from the surgical field and from the patient's body to prevent the following risks.
  1. Injury or damage of patient/surgical staff.
  2. Broken pieces of the Cutting Accessory remaining in the patient's body.
- DO NOT drop Cutting Accessories. If dropped, do not use it.
- The sharp cutting edge of a Cutting Accessory could cause damage to surgical gloves, which can lead to injury, contamination and infection.
- Ensure all Cutting Accessories are properly secured, as mentioned in the Operation Manual of the Attachment/Handpiece.
- Be sure to check there are no abnormal vibration, noises and overheating by activating the handpiece after connecting a Cutting Accessory to the Attachment/Handpiece. If the above abnormal conditions are detected, stop using the Attachment and motor, and contact your Authorized NSK Dealer.
- Be sure to use an appropriate Cutting Accessory to the Attachment/Handpiece (Refer to each Attachment/Handpiece operation Manual).
- While Cutting Accessory is in use, ensure the Cutting Accessory does not make contact with implants (except "Primado Metal Cutting Carbide Bur" and "Primado Metal Cutting Diamond Disc").
- Operators and all others in the area must wear a mask, rubber gloves and protection eye glasses when operating the Attachment/Handpiece.

## CAUTION

- Do not touch the Cutting Accessory during operating, otherwise injury may occur.
- An operating Cutting Accessory could catch hold of drapes such as operating table drapes, which may cause injuries and may also damage the Cutting Accessory and the system. Therefore, under no circumstances should an operating Cutting Accessory be allowed to come in contact with drapes.
- Store in a place without any extreme loads avoiding direct sunlight, high temperatures and humidity.
  - \*1. Appropriate temp; 15 - 32°C
  - \*2. Appropriate humidity; 25 - 75%
- Check the expiry date before use. DO NOT use after the expiry date has passed.
- Protect the region of the wound from the metal tips when using the "Primado Metal Cutting Carbide Bur" and "Primado Metal Cutting Diamond Disc".
- Hold implant parts firmly to prevent debris from flying while cutting the implant. Also, use large amounts of irrigation to cool the cutting surface while cutting.

### 3 Symbols

Manufacturer.

Conforms to CE European Directive of "Medical device directive 93/42/EEC".

Consult operation instructions.

Temperature limitation.

Do not reuse.

Use by.

Keep Dry.

**REF**

Catalogue number.

**STERILE**

Sterilized using irradiation.

**EC**

**REP**

Authorized representative in the European community.

**Rx Only**

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

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**NSK Europe GmbH**